

PMA 2020-S Adapters

For Use Only With Pilling Weck Metal Jackson Improved Tracheostomy Tubes - Sizes 4, 5 & 6 or Equivalent

For use only with the PMV 2020 Tracheostomy Speaking Valve

Contents: Replacement PMA 2020-S (JIT 4-6) Adapters

Quantity: Three (3)

WARNING: Do not use in-line with a ventilator.

Reorder No. PMA 2020-S

Passy-Muir Inc.

website: www.passy-muir.com

email: info@passy-muir.com

Lot No.



PMA 2020-S Adapters

Directions For Use:

1. *Stabilize the tracheostomy tube with one hand.*
2. *With the other hand, firmly press the PMA 2020-S Adapter onto the hub of the tube.*
3. *The notch of the adapter should be lined up with the ball lock of the tracheostomy tube.*
4. *Place the PMV per manufacturer's instructions (see Instruction Booklet PMV 2020).*
5. *Detach the PMV per manufacturer's instructions prior to removal of the PMA 2020-S Adapter.*
6. *Remove the PMA 2020-S Adapter from the hub of the tube by grasping the adapter from the center opening and gently pulling it off of the hub.*
7. *Clean daily with mild soap and warm water.*

Passy-Muir Inc.

Irvine, CA 92612 • (949) 833-8255 • (800) 634-5397

