

Daily Cleaning of One-Way Passy Muir Valve (PMV)



PMV 007



PMV 2001



PMV 2000

1. Swish:

- Pour a few drops of mild liquid dishwashing soap (soap with no lotion or fragrance) into a cup.
- Add lukewarm (not hot) water.
- Swish the valve in the water for about 10 seconds.

2. Rinse:

Rinse the valve thoroughly in lukewarm (not hot) running water.

3. Dry:

Allow the Passy Muir Valve to air dry on its side.

NOTE:

- Do not place in the storage container to dry.
- Do not use a blow dryer to heat or dry.

4. Store:

After drying, place the valve in the storage container and close the lid for storing overnight.